

ERT KERRISON RONGEUR SYSTEM ERT KERRISON RONGEUR SYSTEM

ergonomic line

NEW

360° DREHBAR
360° ROTATION



ERT = EIGHTFOLD - ROTATING - TAKE-APART
RONGEUR SYSTEM

ERT Kerrison 360° drehbares Rongeur System ERT Kerrison 360° rotating Rongeur System

ergonomic line


Advantage:

- Arm of Surgeon can always remain in the same position



- No need for upbiting, downbiting or side-nibbling Kerrisons any more

Choose from 8 different angles while your hand remains in the same position

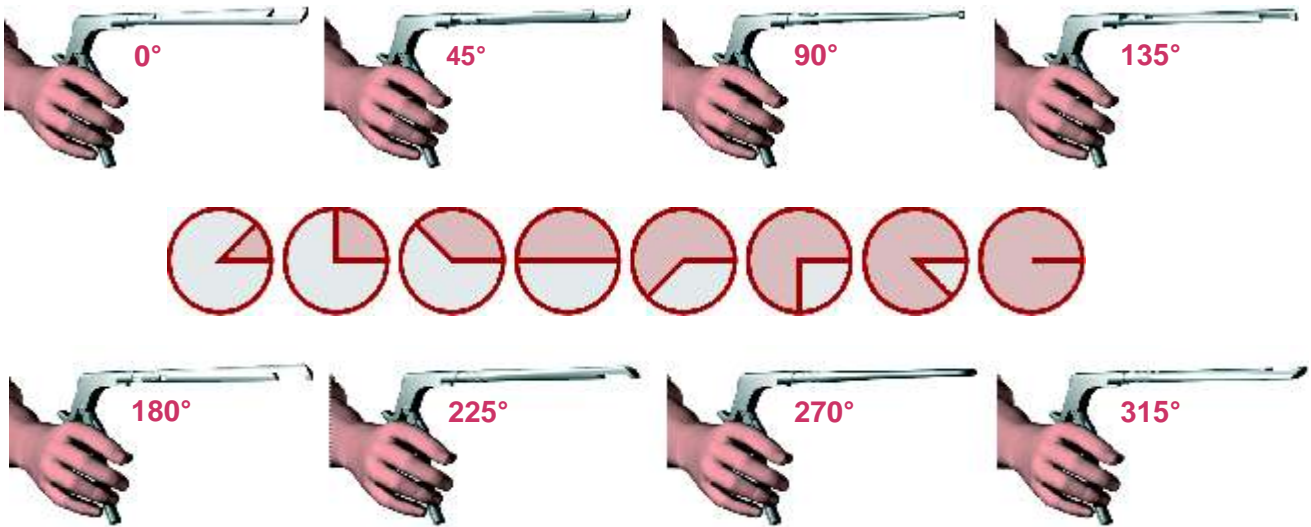

1.

The Kerrison punches are in the primary position. Hold the handle in one hand and place the fingertips of the other hand on the serrations of the lower shaft.

Take care not to hold anything but the serrated part



2.

Pull the serrated part backwards to depress the spring and to enable the rotation of the shaft to the preferred positions. (It's rotating in both directions)



3.

Hold the shaft in the backward position and commence the rotation in the preferred direction.



4.

When you bring the shaft into the desired position, release the spring to set the shaft in the working position. When you hear a click the shaft is fixed and the Kerrison punches are ready to be used.

ERT Kerrison 360° drehbares Rongeur System ERT Kerrison 360° rotating Rongeur System

ergonomic line



ERT Shaft *ergonomic* with TOP throw off and thin food



ERT Shaft *ergonomic* rotating 360° complete 40°
with TOP throw off and thin food

ERT Shaft *ergonomic* rotating 360° complete 90°
with TOP throw off and thin food

Art.-No.	Shaft - length	width of bit
41.133.01TP	18 cm	1 mm
41.133.02TP	18 cm	2 mm
41.133.03TP	18 cm	3 mm
41.133.04TP	18 cm	4 mm
41.133.05TP	18 cm	5 mm
41.135.01TP	20 cm	1 mm
41.135.02TP	20 cm	2 mm
41.135.03TP	20 cm	3 mm
41.135.04TP	20 cm	4 mm
41.135.05TP	20 cm	5 mm
41.137.01TP	23 cm	1 mm
41.137.02TP	23 cm	2 mm
41.137.03TP	23 cm	3 mm
41.137.04TP	23 cm	4 mm
41.137.05TP	23 cm	5 mm
41.139.01TP	25 cm	1 mm
41.139.02TP	25 cm	2 mm
41.139.03TP	25 cm	3 mm
41.139.04TP	25 cm	4 mm
41.139.05TP	25 cm	5 mm
41.141.01TP	28 cm	1 mm
41.141.02TP	28 cm	2 mm
41.141.03TP	28 cm	3 mm
41.141.04TP	28 cm	4 mm
41.141.05TP	28 cm	5 mm
41.143.01TP	30 cm	1 mm
41.143.02TP	30 cm	2 mm
41.143.03TP	30 cm	3 mm
41.143.04TP	30 cm	4 mm
41.143.05TP	30 cm	5 mm

Art.-No.	Shaft - length	width of bit
41.132.01TP	18 cm	1 mm
41.132.02TP	18 cm	2 mm
41.132.03TP	18 cm	3 mm
41.132.04TP	18 cm	4 mm
41.132.05TP	18 cm	5 mm
41.134.01TP	20 cm	1 mm
41.134.02TP	20 cm	2 mm
41.134.03TP	20 cm	3 mm
41.134.04TP	20 cm	4 mm
41.134.05TP	20 cm	5 mm
41.136.01TP	23 cm	1 mm
41.136.02TP	23 cm	2 mm
41.136.03TP	23 cm	3 mm
41.136.04TP	23 cm	4 mm
41.136.05TP	23 cm	5 mm



Handle for
ERT Rongeur
with return function
41.000.02

DR K MEDICAL

www.doctorkmedical.ro office@doctorkmedical.ro 0753.91.87.87 0332.460.299

Instruments &
Implants

DR K MEDICAL

Instrumente si Implanturi

Tel: 0753.91.87.87

Fax: 0332.460.299

office@doctorkmedical.ro

www.doctorkmedical.ro